

Nu.Q® Vet Cancer Test in everyday practice | Reference Lab

Collecting a sample



Draw 2-5ml of blood

From peripheral or jugular vein



Fill EDTA tube

Gently invert 10 times



Centrifuge sample on blood setting

For 10 minutes within 1 hour of collection



Extract plasma

Transfer to non-additive tube
(Avoid disrupting buffy coat)



Send to reference lab

Store sample in fridge until pick-up, and ensure pick-up is within 24 hours

Available through:

IDEXX



Test Code - 8993